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TUBO HIPOSPADIA

Registro na Anvisa: 80020550048

Revisão 02

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Revision 02

INDICATIONS AND INTENDED USE:

The Medicone Hypospadias Tube has been developed to assist in reconstructive surgeries for hypospadias, epispadias, and other malformations of the urethra in which it closes on the underside or upper side of the penis instead of at the tip of the glans. It is also indicated for any other urethral condition requiring the use of a catheter for an extended period. Treatment with the Medicone Hypospadias Tube aims to shape the new urethra and allow drainage of body fluids, restoring the normal aesthetic and functional characteristics of the penis and urethra.

PRODUCT DESCRIPTION AND COMPOSITION:

The Medicone Hypospadias Tube consists of a medical-grade silicone elastomer catheter, with an opening at one end for drainage of body fluids, a tip designed to prevent bladder injury, and a fixation ring.

PRODUCT PRESENTATION:

The product is supplied individually packaged in medical-grade paper with PET/PE laminate.

INSTRUCTIONS FOR USE:

Surgical techniques and procedures are under the responsibility of the physician, who will evaluate the clinical situation on a case-by-case basis.

- Ensure asepsis of the surgical field.
- Follow the hospital protocol for performing the medical procedure.

PRECAUTIONS, WARNINGS, AND OTHER RECOMMENDATIONS

- Medical-Hospital Product – The manufacturer recommends single use only. Reuse is not advised.
- Before use, ensure the packaging is intact and the product is within its expiration date.
- The product must be prescribed and handled only by qualified and trained healthcare professionals.
- Any questions or adverse events should be reported to Medicone. Medicone is not liable for accidents or harmful consequences of any nature resulting from incorrect use of the product.
- Any instrument with a perforating or cutting function must be handled with care to avoid damage to the Medicone Hypospadias Tube.
- Any tear or cut in the Medicone Hypospadias Tube renders it unusable and it must be replaced with a new, intact product.
- The surface of the Medicone Hypospadias Tube must not be contaminated with oil, talc, or dust.

**CONTRAINDICATIONS:**

The product is contraindicated for patients with known sensitivity to sterilizing agents (ETO).

ADVERSE EFFECTS:

Adverse effects inherent to the use of the Medicone Hypospadias Tube include: meatal stenosis; fistula; infection; pain and tenderness; and urethral stenosis in patients with intolerance to the material of manufacture.

Complications: In any surgery involving the introduction of foreign objects into the body, the patient may experience possible complications. In hypospadias repair surgery, complications reported in the medical-scientific literature include: suture dehiscence; skin necrosis; possible infection at the surgical incision site requiring further treatment; hematoma, edema, and lymphedema; urethral fistula requiring new surgical treatment; urethral narrowing requiring dilations or additional procedures; there is no absolute guarantee of cure for hypospadias, and further surgeries may be necessary; and possible scarring with keloid formation (hypertrophic, thick scars). Possible complications associated with the use of the product must be reported and discussed with the patient or their guardian prior to surgery.

STERILIZATION:

The Medicone Hypospadias Tube is supplied sterile from the factory, sterilized by ETO. The sterilization is valid for 36 (thirty-six) months from the sterilization date, which is indicated on the product packaging.

STORAGE:

- Keep the packaging intact and sealed until product use.
- Store at room temperature, protected from heat and moisture.
-

SINGLE USE PRODUCT – DO NOT REPROCESS

The Medicone Hypospadias Tube is intended for single use only. Reuse is not recommended, as it may compromise product performance.

DWELL TIME IN PATIENT:

The Medicone Hypospadias Tube may be used for the period indicated by the physician, who will evaluate each case individually, but it must not exceed 29 days.

DISPOSAL:

After use, dispose of all material in an appropriate container for potentially contaminated waste.

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